




OSCA Obstetric Suction Cup

VERSATILE | EFFICIENT | ECONOMIC

HTKD Medical / Beijing HangTian KaDi Technology R&D Institute

OSCA Obstetric Suction Cup

OSCA Obstetric Suction Cup is a versatile obstetric suction cup used for instrumental extraction procedures. It is suitable for all fetal presentations

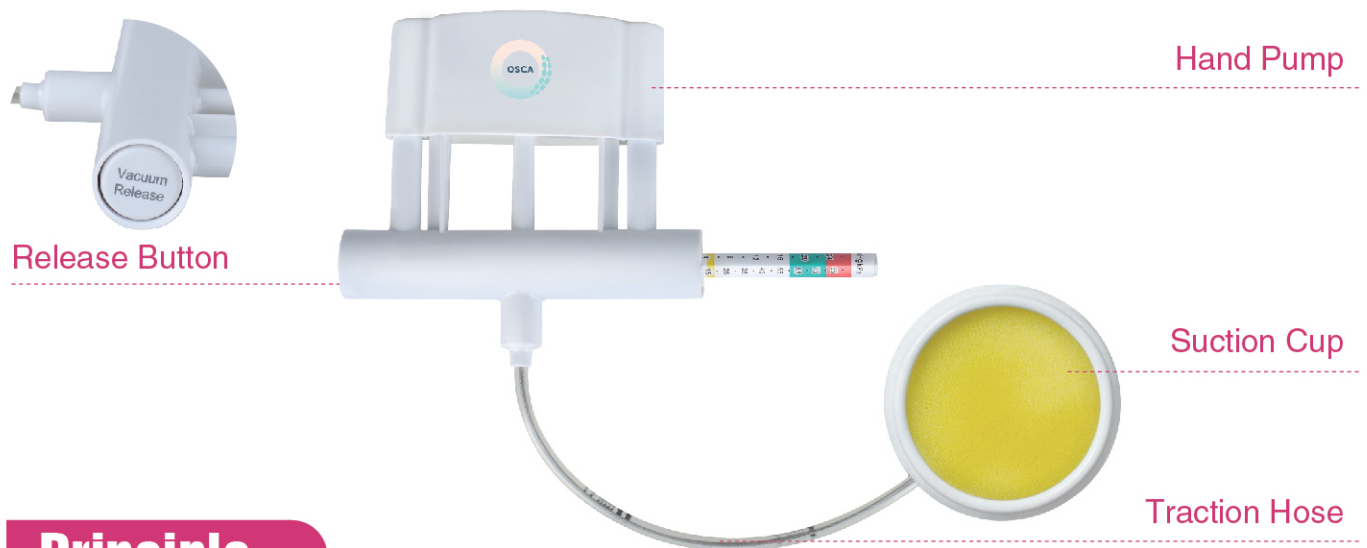

Principle

Simple, Effective and Single-use



OSCA Obstetric Suction Cup is a simple and effective single-use device for instrumental extraction. The traction hose fixed in the center of the suction cup is housed in the tunnel provided for this purpose which allows easy insertion of the suction cup and optimal positioning on the flexion point for all fetal presentations.

Stable and Secure



Once the device is correctly positioned, the hand pump associated with the traction handle generates a vacuum which is applied to the fetal scalp, this has the consequence of stabilizing and securely fixing the head of the fetus to the suction cup.

Precise and Flexible



The graduated vacuum indicator included in the handle gives a precise indication of the vacuum applied to the fetal scalp. The flexible traction cable allows freedom of movement and orientation for effective traction at the rate of maternal contractions.

Features

- The device includes a pull handle associated with the vacuum pump and a colour-coded graduated vacuum indicator for ergonomic and efficient use.
- A push button inserted in the handle allows the vacuum to be released easily.
- The suction cup is fitted with a tunnel where the traction pipe is housed for easy insertion and positioning of the device.
- The vacuum hose contains two steel traction cables and is fixed in the centre of the suction cup. Its flexibility allows freedom of movement and a 180 ° traction angle. It also has two marks (6 and 11cm) which help to assess the insertion distance and the presentation height.
- Latex-free system, DEHP-free (Di-2-ethylhexyl phthalate), BPA-free (bisphenol A).

Usage




Locate the Flexion Point and Calculate the Cup Insertion Distance




Hold and Insert the Cup



Maneuver the Cup Toward and Over the Flexion Point

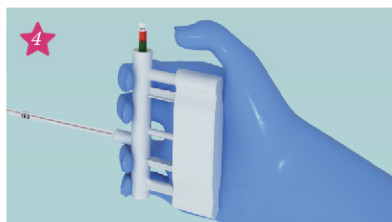

Create Vacuum and Exclude the Maternal Tissue

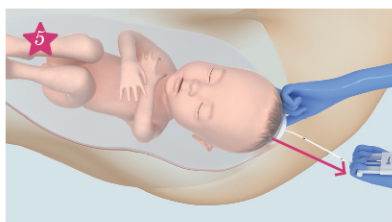

Using the Finger Tip Traction and the Finger/Thumb Technique, Pull Along Axis of the Pelvis

Scan to view
video tutorials



Product Specification



HK-TTQ-A






HK-TTQ-B



HK-TTQ-C

Model	Description	Packaging
HK-TTQ-A	Normal Type Obstetric Suction Cup	· 5PCS/ SMALL BOX, 24.5*18.5*10.5, GW:0.5 KGS · 3 SMALL BOXES/MIDDLE BOX, 33.5*26*19.5,GW: 1.7KGS · 12 MIDDLE BOXES/CARTOON, 71*56*45, GW: 15KGS
HK-TTQ-B	Obstetric Suction Cup with an Extra Pressure Indicator	
HK-TTQ-C	Obstetric Suction Cup for C-Section Use	

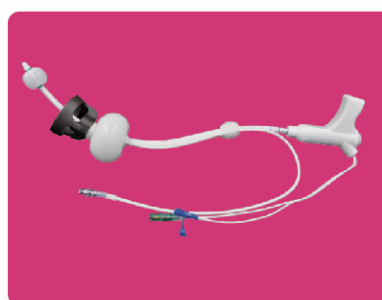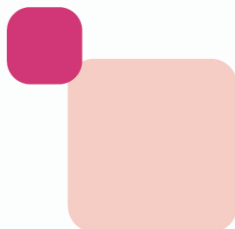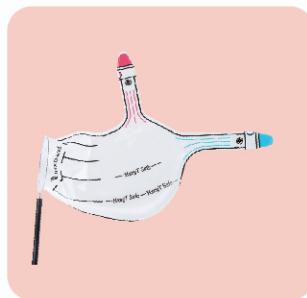

Beijing HangTian KaDi Technology R&D Institute
HTKD Medical



www.htkdmedical.com



Search HTKD Medical



@HTKD Medical



Room 301, Third Floor, Building No.13, No.15 Jing Sheng Nan
Er Street, TongZhou District, Beijing, 101102, China P.R.



CONTACT US

Mr. Jeff Jiang

+86 139 1061 1697

+86 139 1061 1697

Ms. Claire Zhou

+86 185 0035 7781

+86 185 0035 7781



Product information



How to use OSCA